

Customer Information Notice: CIN161

Nanosonics AuditPro™ 2.0 Release

Nanosonics AuditPro™ 2.0 software will be released on 13 March 2023. An automated system update will be pushed to the Cloud Application and the Mobile Application. Existing Users will automatically access the new Cloud Application from 13 March 2023 and will be pushed an update to accept on their Mobile Scanning Device to update the Mobile Application. The enhancements of this software release include:

- **Configurable Department settings with two Solutions available:**
 - **Solution 1: Workflow Compliance Management solution**
 - **Solution 2: Digital Logbook solution**
- **Procedure Logging Workflow Changes**
- **Simplified User Personas**
- **Support of trophon®2 software Version 1.6, Wi-Fi dongle**

Existing AuditPro Customers are encouraged to discuss the new Configurable Department settings with their Nanosonics Representative to determine which solution is best for each department where AuditPro is installed. Existing Customers will be migrated to the Workflow Compliance Management solution by default and will experience an update in the workflow within the mobile scanning device when logging procedures. Customers wishing to move one or more Departments to the Digital Logbook solution can do so by contacting their Nanosonics Representative to schedule this migration with Nanosonics.


Changes have been made to the mobile application for both Department solutions. Users will benefit from reduced logging steps in both Department solution configurations, with the Digital Logbook solution reduced to just 3 steps.

Details of the feature enhancements contained within AuditPro 2.0 can be found on the following pages. If you have questions please contact your Nanosonics Representative or Nanosonics Customer Service.

Feature enhancements contained within AuditPro 2.0

1. Department Configurations

- 2 solutions that can be set at a department level
 - Solution 1: Workflow Compliance Management solution
 - Solution 2: Digital Logbook solution
- Offer additional setting for Solution 2 (Digital Logbook)
 - Option 1: Link cycle event occurred BEFORE procedure
 - Option 2: Link cycle event occurred AFTER procedure

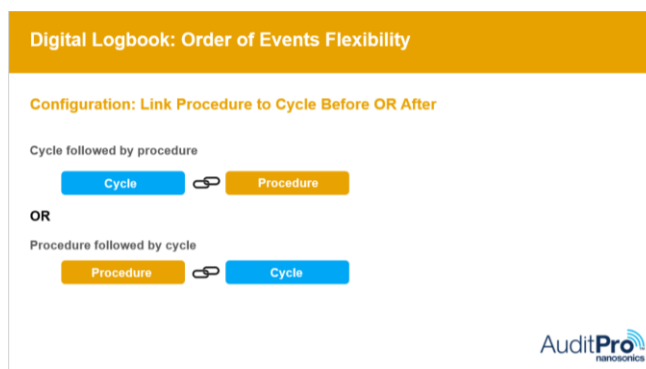


- Department users can only be assigned to a single department

2. Procedure Logging Workflow Options

- The Department Solution associates with the procedure logging workflow. In the cloud application, each mobile scanning device must be assigned to a department to import the relevant workflow onto the mobile scanning device mobile application.
- Solution 1: Workflow Compliance Management (Advanced workflow)

Applicable to the mobile scanning devices assigned to Departments using Solution 1.

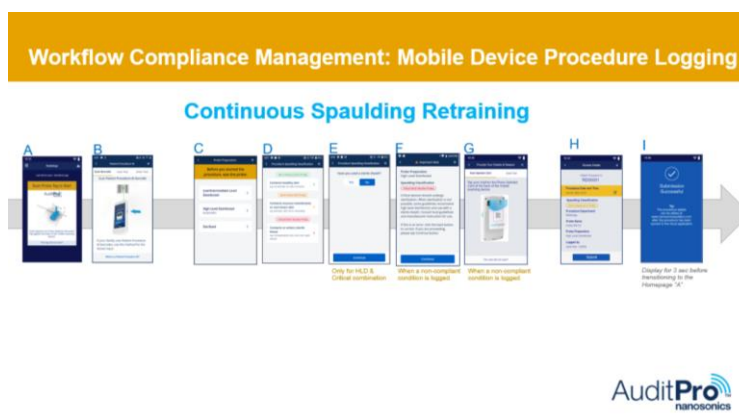


- Solution 2: Digital Logbook (Simplified workflow)

Applicable to the mobile scanning devices that are assigned to Departments using Solution 2.

Skip the probe preparation and Spaulding classification input.

The incorrect probe preparation warning and subsequent operator logging are not applicable for this solution.

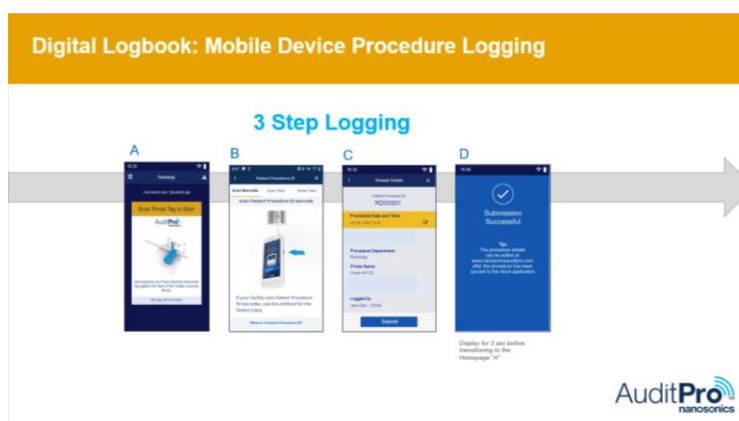


3. Reduced Procedure Logging Steps

- “Log a Procedure” button removed from the mobile application homepage.
- Scan the probe tag (on the homepage) to initiate the procedure logging.
- Procedure date & time is consolidated on the Review page.
- “Acknowledge” button removed after submitting the procedure. The user will be taken back to the homepage after 3 seconds (no user action is required).

4. Updates Homepage Design

- The update includes 4 homepages for 1) Workflow Compliance Management Department Admin 2) Digital Logbook Department Admin 3) Facility Admin and 4) Digital Logbook Operator.

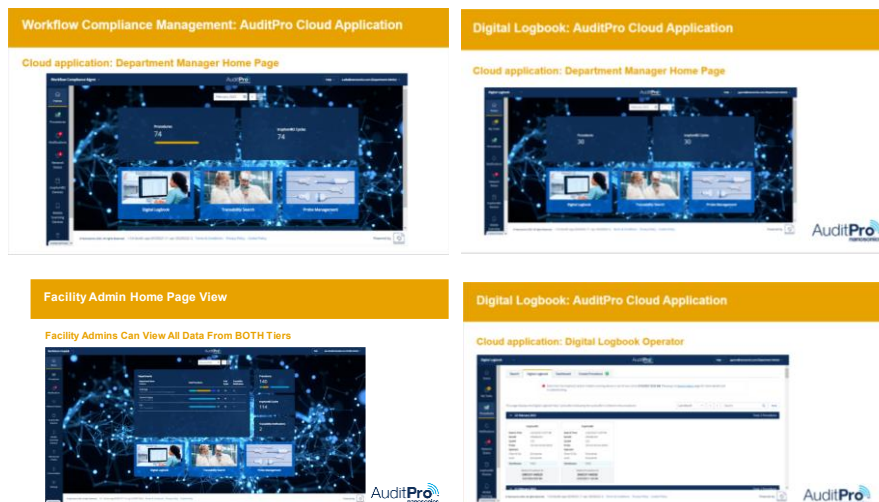

5. Updated Personas

- Removed Facility Guest and Facility Operator.

6. Support trophon2 Software v1.6

- When AcuTrace™ is turned off on the trophon2 device or any of the 3 configurable AcuTrace settings are turned off, an updated AcuTrace™ logic will be triggered.:
 - A notification will be shown in mobile and cloud applications to notify customers.
 - When any of the configurable AcuTrace settings are tuned off AuditPro shall display impacted trophon2 cycle value as “No Information”.