

## Customer Information Notice: CIN176

### AuditPro Release 1.7 Features Summary

Nanosonics AuditPro™ system update release 19 August 2024 will include the following new features:

1. Hang-time protocol: Customers may activate Hang-time protocol via the cloud application settings by toggling the feature on and setting the desired HLD expiry date required to support their facility SOP. Once activated the mobile scanning device (MSD) will notify clinical users if the last HLD cycle conducted on that probe is outside of the configured timeframe.
2. BioMed profile: AuditPro customers have access to a new user persona designed to allow BioMed customers visibility of connected trophon2® devices across their facility. This feature will allow BioMeds to remotely see connectivity status of devices, service due dates, usage patterns of device to assist in scheduling maintenance time, device utilisation and detailed cycle data. BioMed profile may be set for new or existing users.
3. Digital Logbook View options for Facility Admin of Digital Logbook Solution view Cycle After Procedure configuration: This update will allow Facility Admin users to view data in the same order as Department users to assist with error handling and communication.
4. trophon2 connectivity flexibility to support trophon2 V1.7 software update: This feature will remove the requirement for trophon2 devices running software V1.7 and above from being linked to a mobile scanning device to collect trophon2 data. Instead, the trophon2 device running software V1.7 will directly connect to the cloud and will transmit trophon2 device data to the cloud directly. trophon2 devices running software V1.7 and above may now be connected to a separate network to mobile scanning devices to assist with IT management if desired. trophon2 connectivity visibility remains available in the AuditPro Cloud for all devices registered to AuditPro Cloud.

- 5. Additional features: Minor navigation improvements assist Infection Preventionists move between departments and organisations more efficiently to improve user experience; miscellaneous improvements including security maintenance and patches.

|                                                                                                                                                                                                                                              |                                                                                                                                                                                                                                                                                           |                                                                                                                                                                                                                                                                     |                                                                                                                                                                                                                                       |                                                                                                                                                                                                                                                              |
|----------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------|-------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------|---------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------|---------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------|--------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------|
| <b>Nanosonics, Inc</b><br>7205 E 87 <sup>th</sup> Street,<br>Indianapolis, IN 46256 USA<br>T: 1-844-876-7466<br>E: <a href="mailto:info@nanosonics.us">info@nanosonics.us</a><br>W: <a href="http://www.nanosonics.us">www.nanosonics.us</a> | <b>Nanosonics Limited UK</b><br>Ground floor of Unit C1 & C2,<br>Hercules Business Park,<br>Bird Hall Lane, SK3 0UX, UK<br>T: +44 1484 860581<br>E: <a href="mailto:info@nanosonics.co.uk">info@nanosonics.co.uk</a><br>W: <a href="http://www.nanosonics.co.uk">www.nanosonics.co.uk</a> | <b>Nanosonics Europe GmbH</b><br>(EU Representative)<br>Poppenbütteler Bogen 66, 22399,<br>Hamburg, Germany<br>T: +49 40 468 568 85<br>E: <a href="mailto:info@nanosonics.eu">info@nanosonics.eu</a><br>W: <a href="http://www.nanosonics.eu">www.nanosonics.eu</a> | <b>Nanosonics France</b><br>2 route de la Noue 91190,<br>Gif sur Yvette, France.<br>T: 01.64.86.58.59<br>E: <a href="mailto:info@nanosonics.fr">info@nanosonics.fr</a><br>W: <a href="http://www.nanosonics.fr">www.nanosonics.fr</a> | <b>Nanosonics Japan</b><br>8F Yamato Building, 5-27-3 Sendagaya,<br>Shibuya-ku, Tokyo 151-0051 Japan<br>T: +81 (3) 6865 6648<br>E: <a href="mailto:info@nanosonics.jp">info@nanosonics.jp</a><br>W: <a href="http://www.nanosonics.jp">www.nanosonics.jp</a> |
|----------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------|-------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------|---------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------|---------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------|--------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------|