EN Instructions for Use  
trophon® NanoNebulant®

35% hydrogen peroxide

Single use disinfectant delivery cartridge, containing multiple doses of disinfectant.

The following must be read in conjunction with your trophon® EPR / trophon® 2 User Manual. This cartridge is specifically designed for use with the trophon EPR / trophon2 to deliver the correct dosage of disinfectant to ensure high level disinfection (HLD). Each cartridge contains 80mL of 35% w/w hydrogen peroxide solution. Use of disinfectants with differing composition or concentration to that specified may result in damage to the trophon EPR / trophon2, probe and/or reduced effectiveness of the HLD process.

The disinfectant cartridge is intended to be used only in combination with the trophon EPR/ trophon2 device. Once inserted into trophon EPR / trophon2 device, a high level instrument-grade disinfectant system is formed.

Refer to your trophon EPR / trophon2 User Manual to enable the AcuTrace® Technology, if available.

Intended Use

The trophon EPR and trophon2 are designed to provide High-Level Disinfection (HLD) of validated ultrasound probes. High-Level Disinfection is achieved by surface exposure to a controlled dose of hydrogen peroxide mist delivered to a disinfection chamber containing the ultrasound probe.

The trophon EPR / trophon2 system consists of a multiple use instrument combined with a single use disinfectant "trophon NanoNebulant", delivered from a multi-dose cartridge.

**1.Warnings** (Refer to the Safety Data Sheet for more information) **IMPORTANT:** Cartridges are pierced at the top and the bottom side upon insertion of a new cartridge into the trophon EPR / trophon2. A small amount of disinfectant may remain in the cartridge, even when it has been fully used. Follow the instructions carefully to avoid injury.

- Always wear disposable gloves when handling disinfectant cartridges.
- Avoid contact with eyes and skin.
- Do not manually pierce the cartridge.
- Do not attempt to open or use a damaged or distorted cartridge.
- Do not attempt to refill or reprocess the empty cartridge after it has been removed from the trophon EPR / trophon2. This could lead to personnel injury and/or equipment/trophon EPR / trophon2 malfunction or damage.
- Strong Oxidizer: Contact with other materials may cause a fire.
- Corrosive: Harmful if swallowed or inhaled. May cause eye and

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skin burns. Prolonged or repeated skin contact may cause dermatitis.

**Any serious incidents in relation to trophon2 or trophon NanoNebulant should be reported to Nanosonics and / or the authority of your member state.**

2.Emergency Guidelines

- **Spillage:** Always wear personal protective equipment. Turn OFF the trophon EPR / trophon2 at the power switch and at the mains power switch. Dilute with plenty of water and discard according to Occupational Health and Safety Guidelines for your institution.
- **Eye contact:** Rinse immediately with copious amounts of cold water, including under the eyelids for at least 15 minutes. Keep eyes open and do NOT rub eyes.
- **Skin contact:** Flush skin with copious amounts of soap and water for at least 15 minutes. Remove contaminated clothing and shoes. Consult a physician if irritation persists.
- **Ingestion:** Do NOT induce vomiting. If victim is conscious and alert, rinse mouth thoroughly with water. Never give anything by mouth to an unconscious person. Call a poison control centre.
- **Inhalation:** Remove person to fresh air. If symptoms persist, call a physician.

3.Warning Symbol Definitions

|  |                  |  |                |  |                             |
|--|------------------|--|----------------|--|-----------------------------|
|  | Oxidiser         |  | Warning        |  | Corrosive                   |
|  | Caution          |  | Wear Gloves    |  | UN 2014 - Hydrogen Peroxide |
|  | No Air Transport |  | Medical Device |  | Single Use Only             |

4.Other Symbol Definitions

|  |                     |  |                                                             |  |                                    |
|--|---------------------|--|-------------------------------------------------------------|--|------------------------------------|
|  | Batch No.           |  | Product No.                                                 |  | Consult Instructions for Use       |
|  | AcuTrace® RFID Zone |  | Expiry Date                                                 |  | European Authorised Representative |
|  | EU Importer         |  | Conforming to Medical Devices Regulation (EU)2017/745 (MDR) |  |                                    |

5.Inserting the Disinfectant Cartridge

**IMPORTANT:** Do NOT use a distorted cartridge. Always check the expiration date on the cartridge before use.

Cartridge door will automatically open when the cartridge needs replacing. Do NOT force the cartridge door open as this may damage and render the trophon EPR / trophon2 non functional.

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- If AcuTrace is enabled, scan the AcuTrace RFID Zone on the cartridge label (figure 1).
- Remove the cap from the cartridge
- Place the cartridge NECK FIRST into the holder. Do NOT force the cartridge into the holder (figure 2).
- When situated in place correctly, the bottom of the cartridge should be in line with the top of the holder. The door should close smoothly. Do NOT force the door to close.
- The trophon EPR / trophon2 is now ready for use. Once the door is closed, it will automatically lock and cannot be reopened until the cartridge is consumed or purged.
- Cartridge Removal**  
Cartridge removal is required when the cartridge has expired or is empty. This will be indicated by a screen message. The trophon EPR / trophon2 will need to be purged before being transported or when a cartridge has expired to ensure all disinfectant is removed. Refer to the trophon EPR / trophon2 User Manual for further information on purging.

- Follow the trophon EPR / trophon2 onscreen instructions. Wear gloves, the door will open automatically. When directed, the empty cartridge can be lifted out. Avoid touching the pierced areas of the cartridge.
- Do NOT shake or change the orientation of the cartridge. For empty cartridge disposal see section 7 below.
- Replace with a new cartridge. Refer to section 5 above.

7.Disposal of Used Cartridge

Follow procedures of your institution for the disposal of CORROSIVE or OXIDIZING materials or dispose of as per local environmental and government regulatory requirements.

8.Storage

|  |                                                          |  |                         |  |                                                       |
|--|----------------------------------------------------------|--|-------------------------|--|-------------------------------------------------------|
|  | Temperature limits:<br>-10 °C – 25 °C<br>(14 °F – 77 °F) |  | Keep away from sunlight |  | Store this way up as indicated on cartridge packaging |
|--|----------------------------------------------------------|--|-------------------------|--|-------------------------------------------------------|

- Store cartridges in ALL original packaging until use.
- Store cartridge upright as indicated on cartridge packaging.
- Cartridges may distort or explode if not properly stored.

**DANGER**

Harmful if inhaled. Harmful if swallowed. Causes skin irritation. Causes serious eye damage. May cause respiratory irritation. May intensify fire; oxidizer.

Avoid breathing dust/ mist/vapours/spray. Wear protective gloves/ protective clothing. IF ON SKIN: Wash with plenty of soap and water. Dispose of container in accordance with local/regional/national regulation.

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