


# trophon® EPR Wall Mount

## Instructions For Use

	<p>Nanosonics Limited 14 Mars Road, Lane Cove NSW 2066, Australia +61 2 8063 1600 info@nanosonics.com.au</p>	<p>Nanosonics, Inc. 7205 E 87th St Indianapolis, IN 46256, USA 1-844-876-7466 support@nanosonics.us</p>	<table border="1"><tr><td>EC</td><td>REP</td></tr></table> <p>Nanosonics Europe GmbH Poppenbuetteler Bogen 66 22399 Hamburg – Germany +49 40 46856885</p>	EC	REP
EC	REP				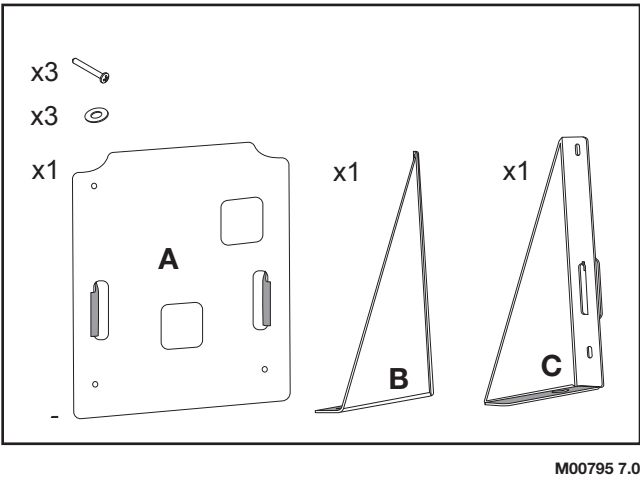

M00795 7.0

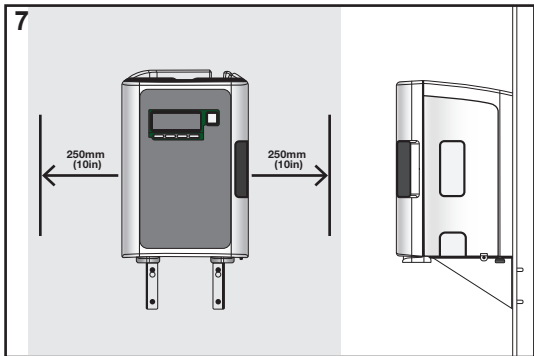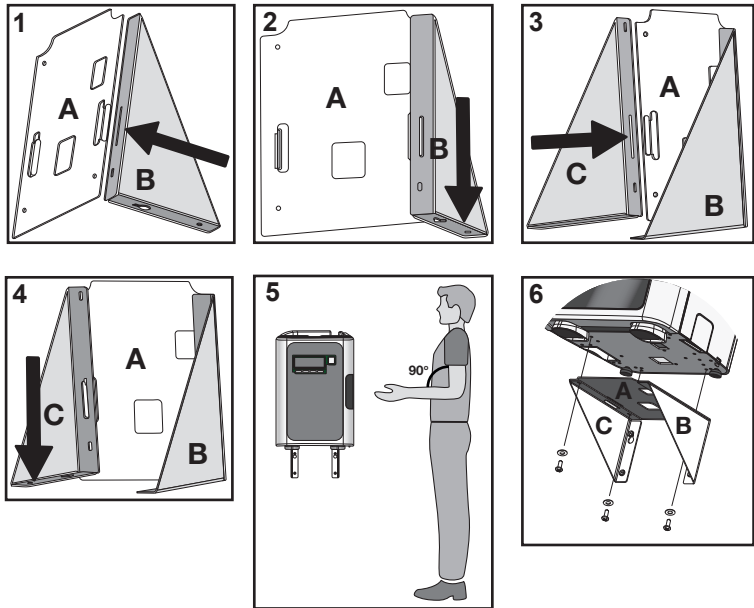www.nanosonics.com.au

## EN

This Instruction For Use has been specifically designed to guide you through the assembly of the trophon EPR Wall Mount and must be read in full BEFORE commencing the assembly.

### Warnings

- ONLY use the Wall Mount specifically designed for use with the trophon EPR.
- Ensure the Wall Mount is assembled and installed by a suitably qualified person from your institution.
- Do NOT install the Wall Mount above any other equipment. Ensure the area underneath the Wall Mount is clear at all times.
- There is potential risk of serious injury and damage to the equipment and device if the Wall Mount is not secured properly with appropriate screws to a suitable wall. The trophon EPR weighs approximately 18kg/40lb.

### 1. Wall Mount Assembly

- a. Refer to images 1, 2, 3 and 4.

**The trophon EPR should be mounted at a height from the floor level to accommodate a range of user heights. Refer to image 5 for a guide to an ergonomically safe work zone for most users.**

**If the trophon EPR has been in use, ensure to run a purge cycle before transferring the device to the Wall Mount. Refer to the trophon EPR User Manual for purging instructions.**

### 2. Installation of the Wall Mount

- a. Install Wall Mount to wall with appropriate screws (not provided) and allow a 250mm (10 in) clearance on either side of the Wall Mount, refer to image 7.

### 3. Securing trophon EPR Device to Wall Mount

- (i). Sit the rear feet of the device on either side of the Wall Mount and gently slide the device backwards until the Wall Mount holes align with the holes on the bottom of the device.
- (ii). Place each star washer over the 3 screws provided.
- (iii). Screw the device to the Wall Mount with the 3 screws, refer to image 6.
- (iv). Refer to image 7 for a correctly mounted trophon EPR device.

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