trophon®


Chemical Indicator

 Nanosonics Limited
7-11 Talavera Road
Macquarie Park
NSW 2113, Australia
+61 2 8063 1600
customerservice@nanosonics.com.au
www.nanosonics.com.au

USA contact:Nanosonics, Inc.
7205 E. 87th Street
Indianapolis, Indiana 46256
1-844-876-7466
customerservice@nanosonics.us
www.nanosonics.us

 Nanosonics Europe GmbH
Poppenbueteler Bogen 66
22399 Hamburg – Germany
+49 40 46856885
customerservice@nanosonics.eu
www.nanosonics.eu

Nanosonics UK Limited
Ground Floor at The Forum
Unit C1 & C2,
Hercules Business Park,
Bird Hall Lane, Stockport,
SK3 0UX, UK
+44 (0) 161 686 3030
customerservice@nanosonics.co.uk
www.nanosonics.co.uk

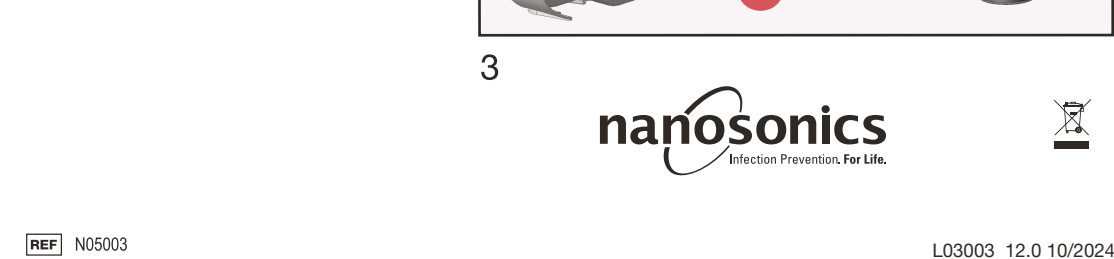REF N05003 L03003 12.0 10/2024

EN - Symbol Glossary DE - Glossar der Symbole FR - Glossaire des symboles NL - Verklarende woordenlijst van symbolen / SV - Symbolförklaring / JP - 記号の説明 / ES - Glosario de símbolos / IT - Glossario dei simboli / NO - Symbolordliste / DK - Symbolordliste / FI - Symbolien selitykset

EN	DE	FR	LOT
EN - Date of Manufacture DE - Herstellungsdatum FR - Date de fabrication NL - Fabricagedatum SV - Tillverkningsdatum JP - 製造日 ES - Fecha de fabricación IT - Data di produzione NO - Produksjonsdato DK - Fabrikationsdato FI - Valmistusajankohta	EN - Manufacturer DE - Hersteller FR - Fabricant NL - Fabrikant SV - Tillverkare JP - 製造者 ES - Fabricante IT - Fabricante NO - Produsent DK - Fabrikant FI - Valmistaja	EN - Product Number DE - Artikelnummer FR - Code produit NL - Productnummer SV - Produktnummer JP - 製品番号 ES - Número de producto IT - Numero prodotto NO - Produktnummer DK - Produktnummer FI - Tuotenumero	EN - Batch No. DE - Chargenz. FR - N° de lot NL - Partijnr. SV - Batchnr. JP - バッチ番号 ES - N.º de lote IT - N. partita NO - Batchnummer DK - Batch nr. FI - Eränumero
 EN - Waste Electrical and Electronic Equipment DE - Abfall elektrische und elektronische Ausrüstung FR - Déchets d'équipements électriques et électroniques NL - Afgedankte elektrische en elektronische apparatuur SV - Elektriskt och elektroniskt avfall JP - 電気・電子機器廃棄物 ES - Eliminación de equipos eléctricos y electrónicos IT - Rifiuti di apparecchiature elettriche o elettroniche NO - Avfall av elektrisk og elektronisk utrustning DK - Afald af elektrisk og elektronisk udstyr FI - Sähkö- tai elektroniikkajätteen hävitys	 EN - Keep Out of Direct Sunlight DE - Vor direktem Sonnenlicht schützen FR - Conserver à l'abri de la lumière directe du soleil NL - Uit de buurt van direct zonlicht houden SV - skyddad från solljus JP - 直射日光を避けること ES - Mantener apartado de la luz directa del sol IT - Tenere al riparo dalla luce solare diretta NO - Oppbevares borte fra direkte sollys DK - Hold væk fra sollys FI - Säilytä poisaa auringonvalosta	 EN - Keep Out of Direct Sunlight DE - Vor direktem Sonnenlicht schützen FR - Conserver à l'abri de la lumière directe du soleil NL - Uit de buurt van direct zonlicht houden SV - skyddad från solljus JP - 直射日光を避けること ES - Mantener apartado de la luz directa del sol IT - Tenere al riparo dalla luce solare diretta NO - Oppbevares borte fra direkte sollys DK - Hold væk fra sollys FI - Säilytä poisaa auringonvalosta	

EN	DE	FR	LOT
EN - European Authorised Representative DE - Bevollmächtigter FR - Mandataire européen NL - Geautoriseerde vertegenwoordiger in Europa SV - Europeisk auktoriserad representant JP - 欧州代理人 ES - Representante europeo autorizado IT - Mandatario per l'Europa NO - Europeisk autorisert representant DK - Europeisk autoriseret repræsentant FI - Valtuutettu edustaja Euroopassa	EN - AcuTrace® RFID Zone DE - AcuTrace® RFID-Zone FR - Zone RFID AcuTrace® NL - RFID-zone AcuTrace® SV - AcuTrace® RFID-zon JP - AcuTrace® RFIDゾーン ES - Zona RFID AcuTrace® IT - Zona RFID AcuTrace® NO - AcuTrace® RFID-soner DK - AcuTrace® RFID-soner FI - AcuTrace® RFID-alue	EN - EU Importer DE - EU-Importeur FR - Importateur UE NL - Importeur in EU SV - Importör i EU JP - EU輸入者 ES - Importador de la UE IT - Importatore UE NO - EU-importør DK - EU-importør FI - Maahantuojat Euroopan Unionin alueella	EN - Expires DE - Verfaltdatum FR - Date d'expiration NL - Utløper SV - Utlögr JP - 有効期限 ES - Caducidad IT - Scadenza NO - Går ut på dato DK - Vervaldatum FI - Vanhenee
 EN - Single use DE - Für den einmaligen Gebrauch FR - À usage unique NL - Voor eenmalig gebruik SV - För engångsbruk JP - 単回使用 ES - De un solo uso IT - Monouso NO - Engangsbruk DK - Enkelt brug FI - Kertäkäyttöinen	EN - Instructions for Use DE - Gebrauchsanweisung FR - Qualification de l'Installation NL - Gebruiksaanwijzing SV - Bruksanvisning JP - 使用方法 ES - De un solo uso IT - Monouso NO - Engangsbruk DK - Enkelt brug FI - Kertäkäyttöinen	EN - Instructions for Use DE - Gebrauchsanweisung FR - Qualification de l'Installation NL - Gebruiksaanwijzing SV - Bruksanvisning JP - 使用方法 ES - De un solo uso IT - Monouso NO - Engangsbruk DK - Enkelt brug FI - Kertäkäyttöinen	 EN - Single use DE - Für den einmaligen Gebrauch FR - À usage unique NL - Voor eenmalig gebruik SV - För engångsbruk JP - 単回使用 ES - De un solo uso IT - Monouso NO - Engangsbruk DK - Enkelt brug FI - Kertäkäyttöinen

EN Instructions for Use
Der trophon® Chemical Indicator is a qualitative chemical indicator designed to monitor the minimum effective concentration (MEC) of the disinfectant during the trophon® EPR / trophon®2 high level disinfection (HLD) process. Only use this product with trophon EPR / trophon2. Please read the trophon EPR / trophon2 User Manual for important information and warnings prior to using the trophon EPR / trophon2 and Chemical Indicator.

Important Information	AcuTrace® :
Use MEC color with enclosed Lot of Chemical Indicators only. Frequency of Use: Use one Chemical Indicator with each trophon EPR / trophon2 HLD cycle. MEC: For the Chemical Indicator to be effective, it must pass the MEC. Refer to the Color Assessment Chart on the packaging for the pass and fail criteria. Pass Criteria: Each HLD cycle is ONLY successful when BOTH: • The color of the Chemical Indicator falls within the pass criteria specified on the Color Assessment Chart (Refer to packaging) AND • The trophon EPR / trophon2 indicates that the cycle is complete. Storage: • Store at room temperature 15°C – 30°C (59°F – 86°F). • Store in a dry and clean environment, out of direct heat and UV light. • Do not store near or expose to chemicals such as sterilising agents, acids, bases, bleaches and other disinfectants.	Refer to the trophon EPR / trophon2 User Manual and on screen instructions to enable the AcuTrace Technology. Sheelf Life/Expiry: • Refer to packaging for expiration date. • Discard any unused Chemical Indicators after the expiration date. General Precautions: • Not intended for use with any other HLD process. • Gloves must be worn when removing the Chemical Indicator from the chamber. • Discard used/exposed Chemical Indicator – do not store for later reference. • Do not use Chemical Indicator after expiration date as shown on packaging. Shipping Information: Not Restricted. The trophon Chemical Indicator is NOT a hazardous product according to the OSHA Hazard Communication Standard 29 CFR 1910.1200. Discard Chemical Indicator packaging once empty.

- Instructions for Use
- If AcuTrace is enabled, scan the AcuTrace RFID zone on the Chemical Indicator packaging when a new pack is used (Figure 1).
 - Remove a trophon Chemical Indicator by pressing out from the perforated sheet (Figure 2) and re-close the packaging.
 - Ensure the Chemical Indicator is RED in color.
 - Place the Chemical Indicator (red side facing up) into the holder located at the bottom of the chamber door (Figure 3).
 - After loading the ultrasound probe, close the chamber door and follow the onscreen messages and trophon EPR / trophon2 User Manual to initiate the HLD cycle.
 - When the HLD cycle is complete, wear gloves and remove the Chemical Indicator from the trophon EPR / trophon2.
 - Visually inspect the Chemical Indicator for color change and verify this against the Color Assessment Chart (see packaging). The Chemical Indicator result and trophon EPR / trophon2 screen message should be interpreted in conjunction with each other. Refer to Table 1 for further instructions
 - Discard the used Chemical Indicator.

Following exposure to the trophon EPR / trophon2 HLD process, the trophon Chemical Indicator will change color, indicating a passed or failed cycle:

Table 1 . Color Assessment Instructions (Refer to chart on packaging)

Indicator Color	Screen Message	Process Pass/ Fail	Instructions
More yellow than MEC	HLD Cycle Complete	✓ Pass	Pass: • Wear gloves and discard used Chemical Indicator. • Wipe the probe with a dry, clean, single-use, low lint cloth. • The probe is now ready for use or storage.
More yellow than MEC	HLD Cycle Failed	✗ Fail	Fail: • Wear gloves and discard used Chemical Indicator. • Insert new Chemical Indicator. • Repeat the trophon EPR / trophon2 HLD process.
MEC or more red	HLD Cycle Complete	✗ Fail	Fail: • Wear gloves and discard used Chemical Indicator. • Insert new Chemical Indicator. • Repeat the trophon EPR / trophon2 HLD process.
MEC or more red	HLD Cycle Failed	✗ Fail	Fail: • Wear gloves and discard used Chemical Indicator. • Insert new Chemical Indicator. • Repeat the trophon EPR / trophon2 HLD process.

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USA contact: Nanosonics, Inc.
REP Nanosonics Europe GmbH