trophon[®]

Wall Mount 2

Nanosonics Limited
7-11 Talavera Road
Macquarie Park
NSW 2113, Australia
+61 2 8063 1600
customerservice@nanosonics.com.au
www.nanosonics.com.au

USA Contact:
Nanosonics, Inc.
7205 E. 87th Street
Indianapolis, Indiana 46256
1-844-879-7466
customerservice@nanosonics.us
www.nanosonics.us


Nanosonics UK Limited
Ground Floor at The Forum
Unit C1 & C2, Hercules Business Park,
Bird Hall Lane, Stockport,
SK3 0UX, UK
+44 (0) 161 686 3030
customerservice@nanosonics.co.uk
www.nanosonics.co.uk

Made in New Zealand






REF N05009

nanosonics

Infection Prevention. For Life.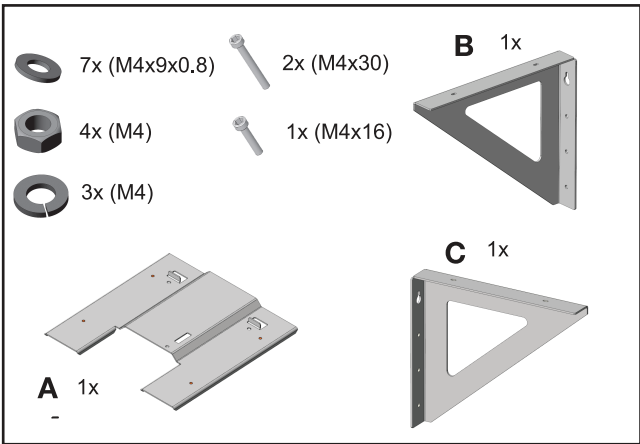

L03047 5.0

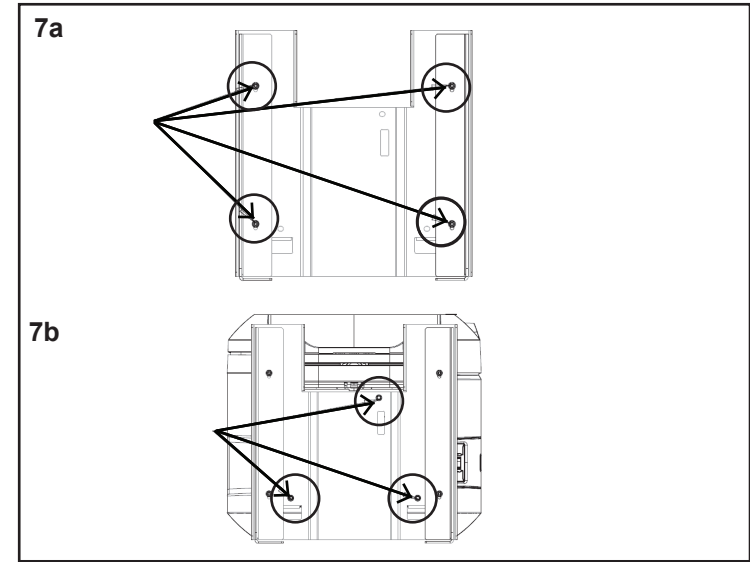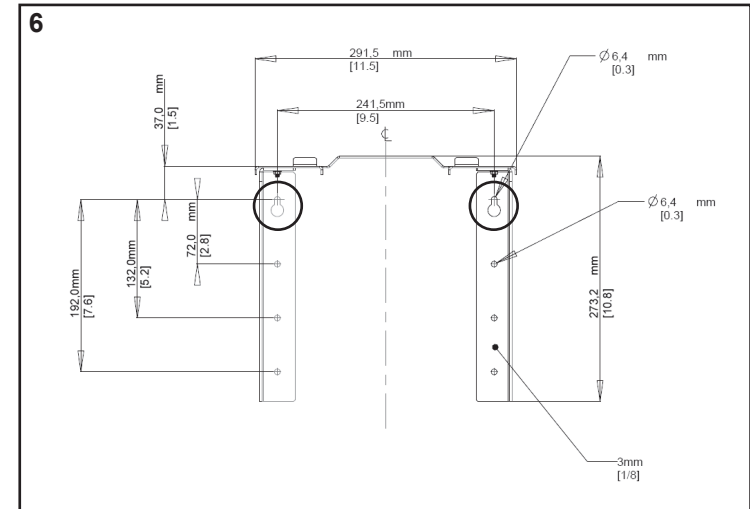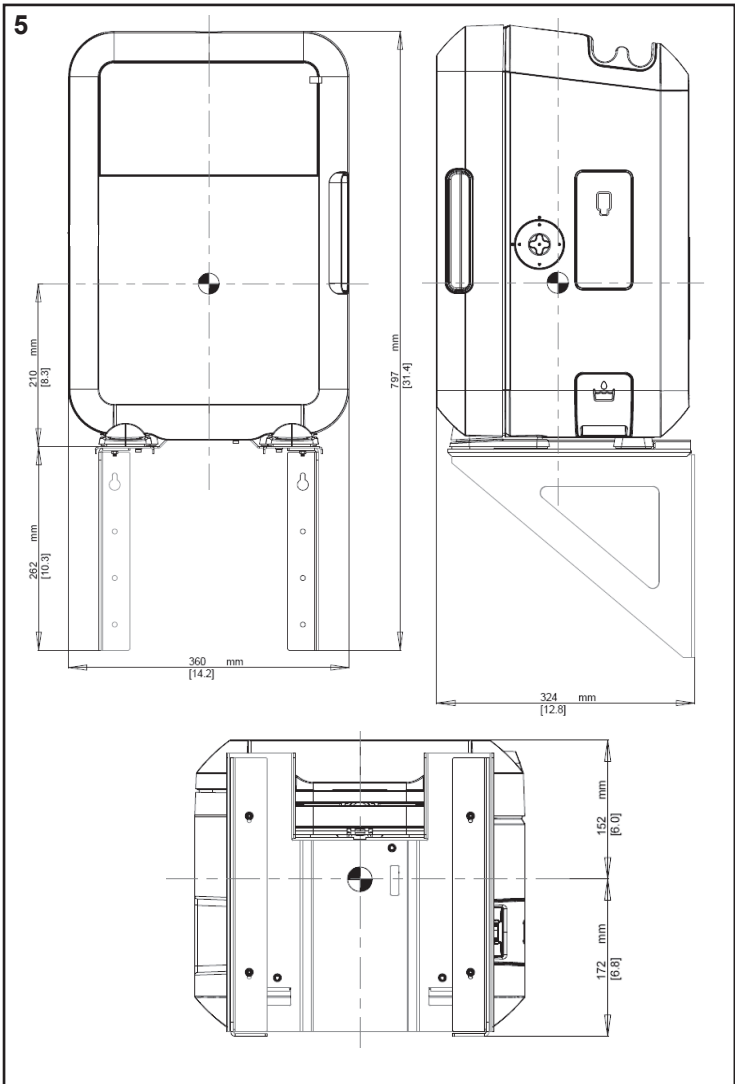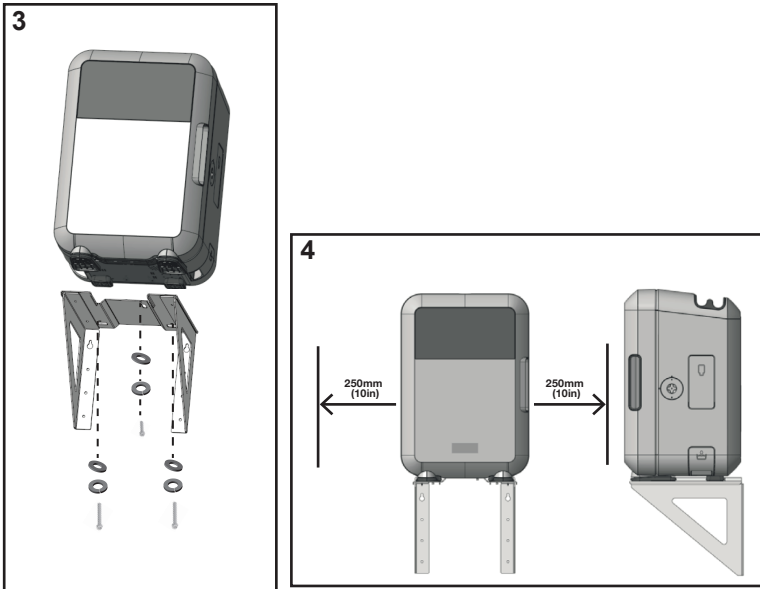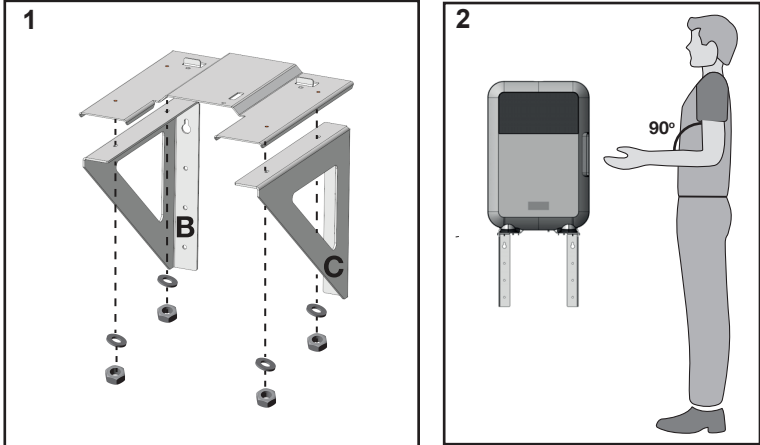Instructions for Use

trophon[®] Wall Mount 2 is intended to mount only the trophon[®] 2 on the wall, in a health care facility. These instructions for use have been specifically designed to guide you through the assembly of the Wall Mount 2 and must be read in full BEFORE commencing the assembly.

Warnings

- ONLY use the specifically designed trophon Wall Mount 2 for use with the trophon2.
- Ensure the Wall Mount 2 is assembled and installed by a suitably qualified person from your institution. Anchorings, screws and the applicable tightening torque for wall mounting should be defined by a structural engineer.
- Do NOT install the Wall Mount 2 above any other equipment and ensure the area underneath the Wall Mount 2 is clear at all times.
- There is potential risk of serious injury and damage to the equipment and trophon2 if the Wall Mount 2 is not properly secured with appropriate screws to a suitable wall. The trophon2 weighs approximately 22kg/ 48.5lb.
- Do NOT install the Wall Mount 2 with the trophon2 attached.

Refer to figure 5 for the trophon2 centre of gravity when mounted to the Wall Mount 2, inclusive of the approximate weights: Wall Mount 2 (3.8kg/ 8.4lbs), probe with connector and cabling (0.9kg/ 2lbs) and trophon2 (22kg/ 48.5lb).

Specifications for parts provided:

Total	Model	Material / Description	Tightening Torque Required
Wall Mount 2 top plate (A) to supports (B and C) (Refer to Figure 7a):			
4	M4	Nylon insert lock nuts	1.5Nm
4	M4 x9x0.8	304 Stainless steel flat round washer	N/A
trophon2 to Wall Mount 2 (Refer to Figure 7b):			
1	M4 x16	304 Stainless steel socket head cap screw	0.8Nm
2	M4 x30	304 Stainless steel socket head cap screw	N/A
3	M4	Zinc plated spring washer	
3	M4 x9x0.8	304 Stainless steel flat round washer	

Total	Material / Description	Thickness at Anchor Points
1	Electro galvanised steel sheet, JIS G 3313 Wall Mount 2 supports (parts B and C)	3mm

1. Wall Mount 2 Assembly

- Refer to Figure 1 and Table A.
- The trophon2 should be mounted at a height from the floor level to accommodate a range of user heights. Refer to Figure 2 for a guide to an ergonomically safe work zone for most users.
- If the trophon2 has been in use, ensure to run a purge cycle before transferring to Wall Mount 2. Refer to the trophon2 User Manual for purging instructions.

2. Installation of Wall Mount 2

- If mounting to a drywall (plasterboard, gypsum panel etc.) the top mount of each bracket must be anchored to the wall stud or cross bar. Refer to Figure 6 for anchor hole locations/ dimensions, and Table B.
- A minimum of four fasteners are required and must include the top two positions (circled on Figure 6).
- Maximum equivalent bolt tensile load at wall interface =600N (each).

a. Install Wall Mount 2 to the wall with appropriate screws (not provided) and allow a 250mm (10in) clearance on either side of the Wall Mount 2, refer to Figure 4.

3 Securing trophon2 to Wall Mount 2

- Lift trophon2 (using two people) and sit the feet on to the Wall Mount 2. Gently slide backwards until the Wall Mount 2 holes align with the holes on the bottom of the trophon2.
- Place each spring washer and then flat washer over the three screws provided.
- Screw the trophon2 to the Wall Mount 2 (see Table A) using the two long screws at the rear and the short screw at the front. Refer to Figure 3.
- Refer to Figure 4 for a correctly mounted trophon2.
- Paint or lacq mark the wall screws and three screws attaching trophon2 to the Wall Mount 2 to distinguish their relative rotational position for future reference.
- Complete the checklist in Table C and file in a safe location such as a disinfection logbook for future reference.

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trophon2 SN:		Date of installation:	/	/	Room location:	
Brand and model of anchorings / screws used for wall attachment:						
Tightening torque(s) applied to above anchorings / screws:						
Name of structural engineer that defined anchoring and sign off:						
Tightening torque applied to screws attaching trophon2 (if different from table A):						
Paint or lacq mark applied:		☐ YES	☐ NO			

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Nanosonics Europe GmbH