Cart UPS Bracket

Instructions for Use

REF N00110

ENGLISH.....	3
DEUTSCH.....	8
FRANÇAIS.....	15
日本語.....	21
ITALIANO.....	29
NEDERLANDS.....	37
ESPAÑOL.....	45
NORSK.....	53
SUOMI	61
DANSK	69
SVENSKA	64


- For trophon EPR Cart (N00039) use only the Uninterrupted Power Supply (UPS) **APC Smart-UPS 750I (SMT750I)**. No other UPS is recommended for use with the trophon EPR Cart.
- For trophon Cart (N05010) use only a UPS that meets the specifications outlined in the Technical Specifications table of this Instruction For Use (IFU), such as the **APC Smart-UPS 750I (SMT750I)**.
- The UPS should not be used for the operation of High Level Disinfection (HLD) and purge cycles with a trophon EPR/trophon2 device, without main connections.
- The UPS should be centred on the Cart UPS bracket for maximum retention. If the UPS is not positioned as per these instructions, there is risk it will not be properly retained during movement and risk of injury.
- UPS vents should not be blocked at any time. Avoid obstruction to ventilation and cooling airflow to the UPS after mounting.
- Follow cable handling as per this IFU.
- Follow the UPS manufacturer's IFU for proper installation and operation, in addition to this IFU.
- Contact your Nanosonics representative if you have any issues with using the Cart UPS Bracket.

Installation Guide

1. If installing the Cart UPS Bracket to an existing cart, dismount the trophon device from the cart.
2. To separate the wheelbase from the cart stem, turn the cart on its side and use a **5 mm Allen key** to remove the four bolts and spring washers from the underside of the cart wheelbase (see Figure 1).

To install a Cart UPS Bracket to a new cart, proceed to **Step 3** for trophon EPR Cart & **Step 4** for trophon Cart.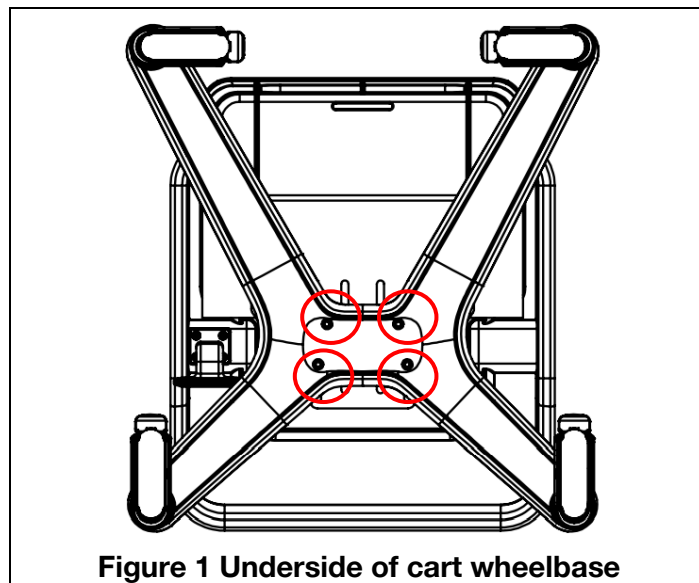


3. **For trophon EPR Cart (N00039)**, place the Cart UPS Bracket on top of the wheelbase, ensuring the vertical back of the bracket is located on the **rear** of the cart (Figure 2a). Align the two slots on the base of the bracket with the 4 bolt holes located on the wheelbase (see Figure 3a).
4. **For trophon Cart (N05010)**, place the Cart UPS Bracket on top of the wheelbase, ensuring the vertical back of the bracket is located on the **front** of the cart (Figure 2b). Align the two slots on the base of the bracket with the 4 bolt holes located on the wheelbase (see Figure 3b).
5. Allow sufficient room between the cart stem and vertical back of bracket to fit UPS:


For trophon EPR Cart: approx. 14 cm.

For trophon Cart: Min. 8 cm, Max. 14 cm


Add and align the cart stem on top of the bracket (see Figure 2). For complete cart assembly, refer to Figure 5.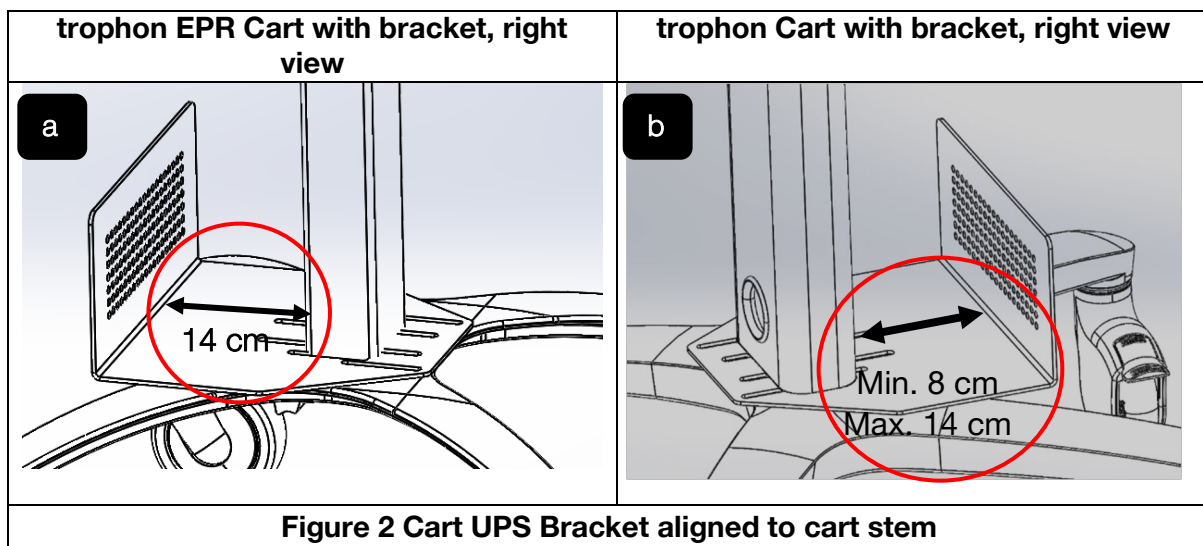


Note: Cart UPS bracket installed to the **back** of trophon EPR Cart and to the **front** of trophon Cart.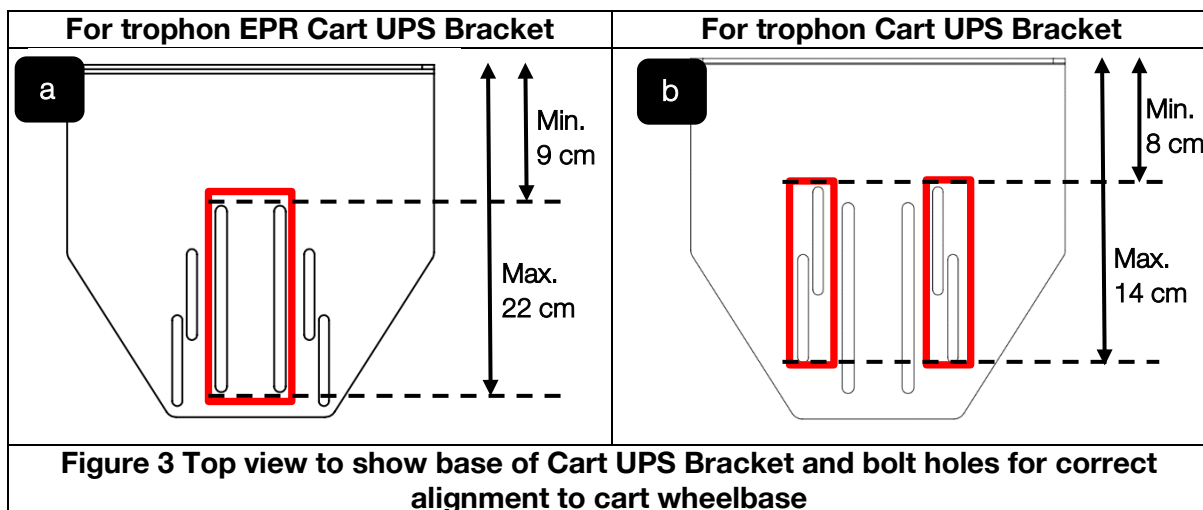


6. Add the 4 spring washers and bolts under the wheelbase of the cart, ensuring they align and pass through the wheelbase, Cart UPS Bracket and cart stem and screw using the Allen key until secure. Do not fully tighten until the UPS has been added.
7. Add the UPS to the bracket (Figure 4) with the power sockets aligned to the trophon device's as per Figure 4 when mounted onto the cart. Ensure the UPS is centred on the bracket. Secure the UPS tightly between the vertical back of bracket and stem by adjusting the bracket as necessary. Once the UPS is tightly secured, tighten the bolts under the wheelbase using the Allen key.

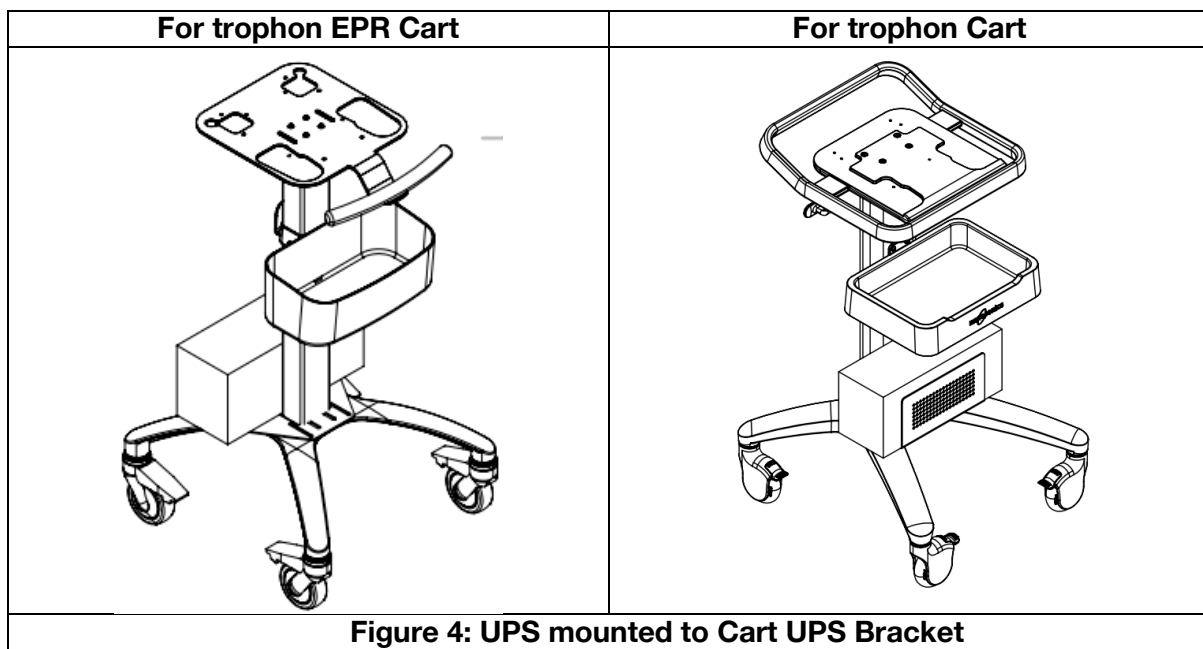8. If installing the bracket to a new cart, refer to the trophon EPR Cart/ trophon Cart IFU to either install the trophon device to the cart's top plate, or to continue assembling the cart.
9. Connect the power cables, as per the UPS manufacturer's instructions. Secure any loose cables to avoid the chance of them being caught under the wheels of the cart during transport, or any trip hazards. Utilise the cavity within the cart stem which is designed to feed cables through (Figure 5).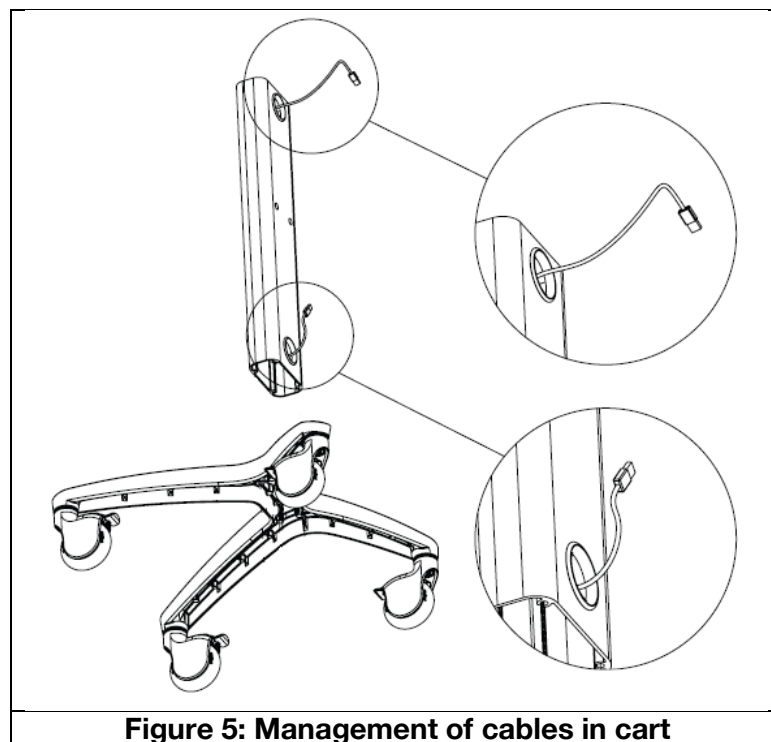

10. Figure 6 shows a correctly installed UPS with the trophon EPR and trophon2 Carts, right view of carts.

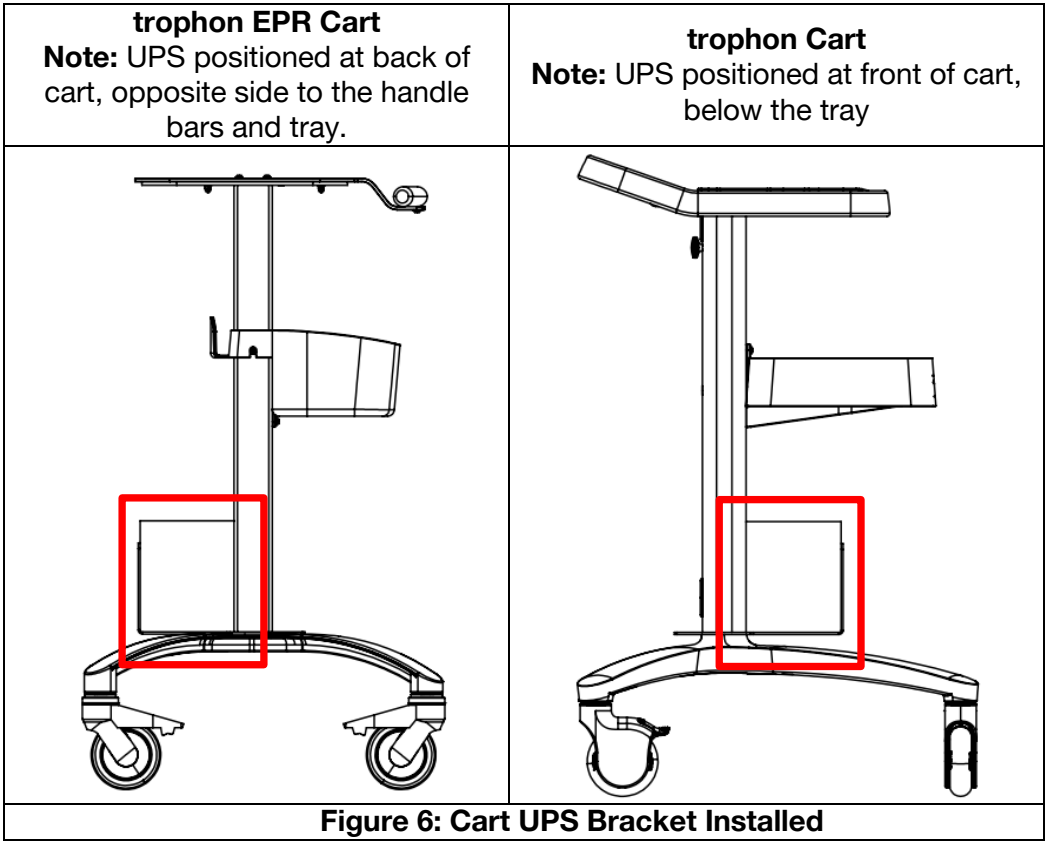N00110 trophon Cart UPS Bracket Technical Specification

trophon EPR Cart can only be used with APC Smart-UPS (SMT750I). trophon Cart is also compatible with APC Smart-UPS (SMT750I). Alternate UPS brands can be used with trophon Cart only if the specifications listed below are met.

Electrical Specification	Output power capacity: 500 Watts/750VA or greater. Waveform Type: Sine wave. For Japan, US, Canada and Mexico Nominal Output Voltage: 100, 120 V AC Nominal Input Voltage: 100, 120 V AC For Rest of World Nominal Output Voltage: 230 V AC Nominal Input Voltage: 230 V AC
Mechanical Specification	Minimum width: 80 mm Maximum width: 140 mm Maximum height: 290 mm Maximum depth: 450 mm Maximum weight: 16 kg

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Nanosonics, Inc.
Nanosonics Europe GmbH
Nanosonics UK Limited